



WPC Compressor™

Casting Procedure:

The WPC Compressor™ is a custom brace that requires a cast of the affected area circumferentially. You may use either plaster or fiberglass wrap. (We prefer fiberglass) The wrap should extend from just below the sternal notch to 3" below xiphoid.

It is also necessary that the order form be completed in order to ensure proper fabrication and a successful fit. If there are any questions regarding the order form, please call Greg Bauer or Brian Whitacre at 1-888-552-2555.

Wearing Protocol:

The brace is to be worn at all times (24/7.) We ask the patient to wear it during the day, during physical activities and even to sleep. The brace will work only while it is worn. The more the child wears the brace, the better effect it will have. The average time required for the patient to wear the brace is about one year.

LCode:

The only code applicable at this time is a L1499.

(Chest Compressor, dynamic, anterior and posterior sagittal control, produces intracavitary pressure, anterior extends from three inches superior to pectus prominence to three inches inferior to pectus prominence, rigid circumferential frame with rigid pads, anteriorly and posteriorly, custom fabricated, molded to patient model.